



Drug Coated Coronary Balloon Catheter


 INNOVATION FOR CARDIO HEALTH



● Features

Drug

- Classic coating combination of Paclitaxel and Iopromid
- Drug concentration $3\mu\text{g}/\text{mm}^2$

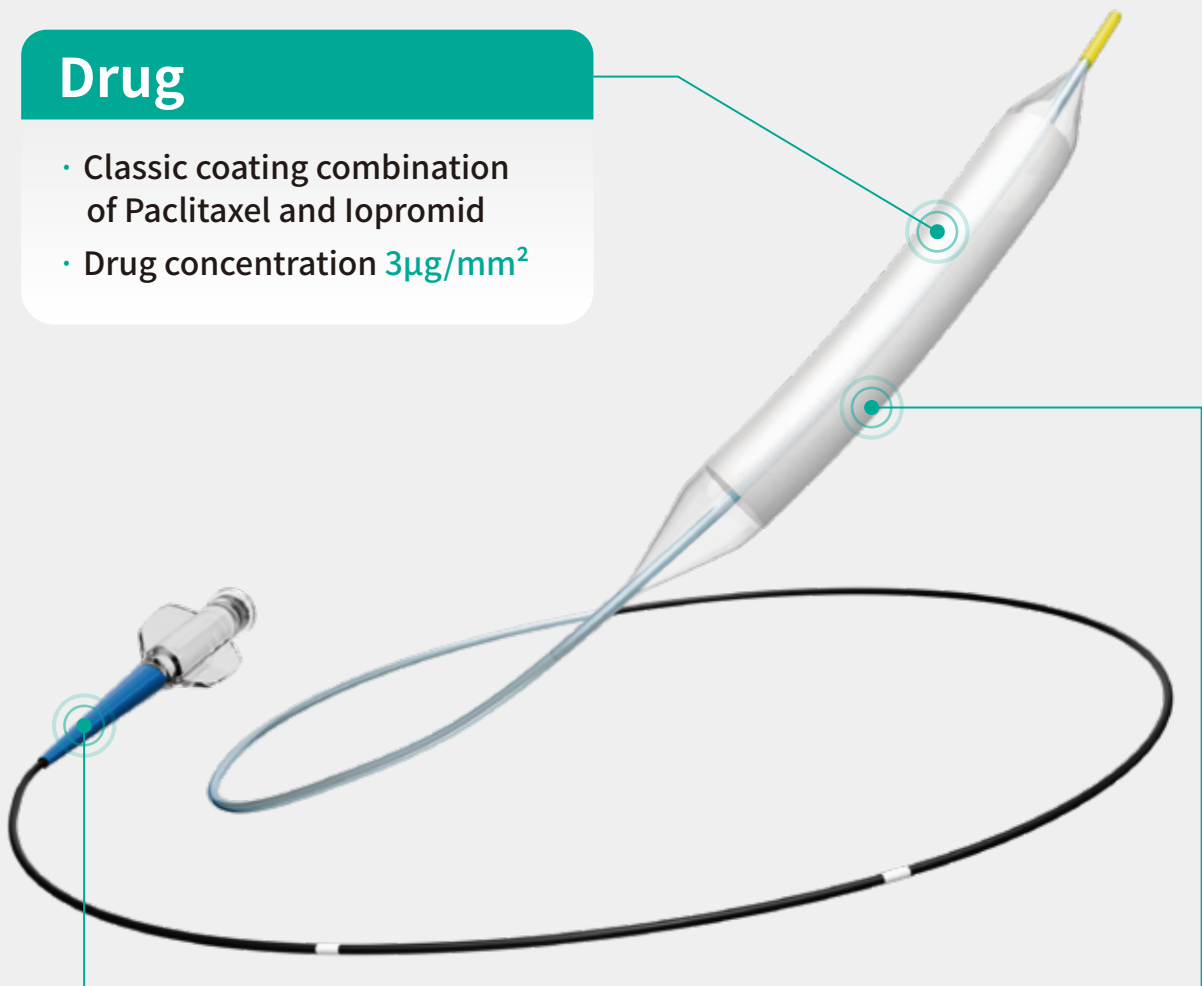

Push rod

- Exceptional delivery system enhances the pushability and handling during operation

High-density coating

- Uniform and dense
- Stabilization and firmness to reduce drug loss during delivery

● Patented coating technology


Chinese Invention Patent-Patent Number : ZI 2016 1 0971410.X

Innovative **ultrasonic spraying technology**

High densely drug particles, superior coating density and stabilization

Multiple coating technology

Reduced drug crystal size compared with single coating

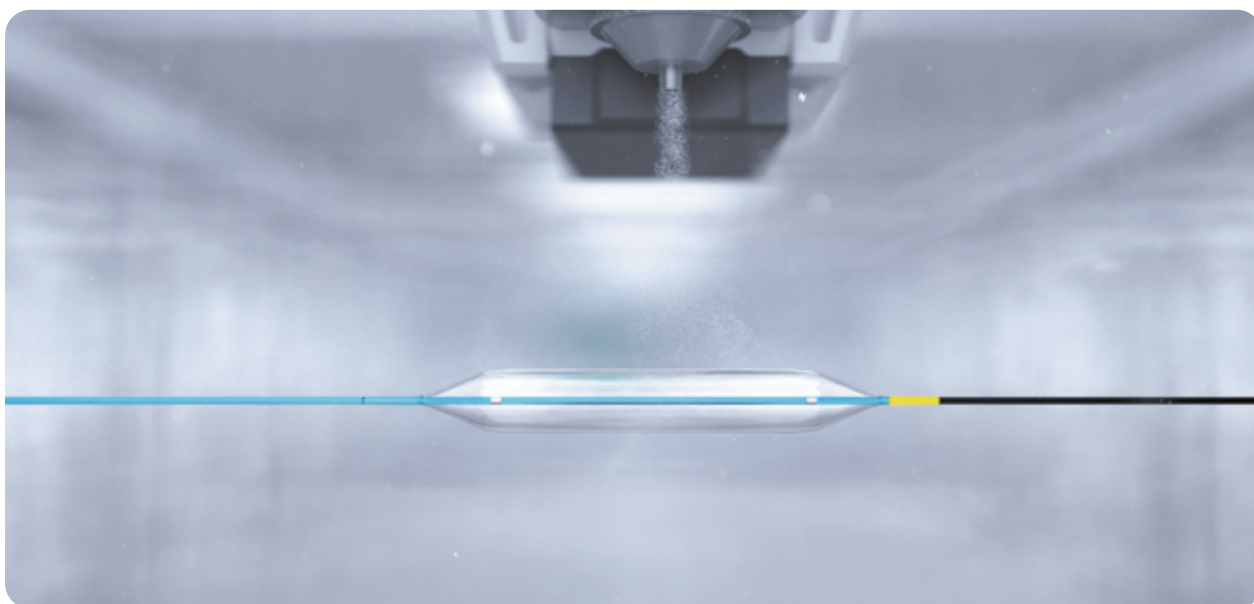

● Technical data

Usable catheter length



1400mm

Guiding compatibility



6F

Balloon diameter




2.0-4.0mm

Recommended inflation time



30s

Guidewire compatibility

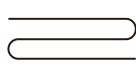

0.014"

Balloon length



10-30mm



Ordering information

Diameter (mm)	Length(mm)				
	10	15	20	25	30
2.00	DP0120010	DP0120015	DP0120020	DP0120025	DP0120030
2.25	DP0122510	DP0122515	DP0122520	DP0122525	DP0122530
2.50	DP0125010	DP0125015	DP0125020	DP0125025	DP0125030
2.75	DP0127510	DP0127515	DP0127520	DP0127525	DP0127530
3.00	DP0130010	DP0130015	DP0130020	DP0130025	DP0130030
3.50	DP0135010	DP0135015	DP0135020	DP0135025	DP0135030
4.00	DP0140010	DP0140015	DP0140020	DP0140025	DP0140030

Compliance data

Pressure (atm)	Average Balloon Diameter (mm)						
	2.00	2.25	2.50	2.75	3.00	3.50	4.00
4	1.88	2.08	2.38	2.66	2.90	3.00	3.86
6	1.94	2.17	2.44	2.72	2.95	3.46	3.93
8(NP)	2.00	2.25	2.50	2.75	3.00	3.50	4.00
10	2.08	2.37	2.55	2.83	3.08	3.56	4.07
12	2.12	2.45	2.60	2.89	3.15	3.62	4.14
14	2.15	2.51	2.64	2.97	3.21	3.70	4.21
16(RBP)	2.19	2.58	2.69	3.03	3.28	3.79	4.28
18	2.23	2.63	2.73	3.07	3.34	3.86	4.34
20	2.28	2.69	2.78	3.12	3.40	3.92	4.40

This file is for healthcare professionals only.



SHENQI MEDICAL

Shanghai Shenqi Medical Technology Co., Ltd.

Floor 1 & Room 601, Building 8, 200 Niudun Road, China (Shanghai) Pilot Free Trade Zone

201203 Shanghai PEOPLE'S REPUBLIC OF CHINA

Tel: +86-21-31300065 Fax: +86-21-50860367

Service email: customerservice@sqmedical.com Website: [http:// www.sqmedical.com](http://www.sqmedical.com)

All cited trademarks are the property of their respective owners in respective countries.

These devices shall only be used for curing diseases on prescription by a licensed physician. Indications contraindications, warnings, potential adverse events and directions for use can be found in instructions for use and product labeling supplied with each device. These devices are prohibited to promote, distribute or sell in any countries or regions where the registration certificates or any other licenses are not obtained. This brochure is not intended for use in aforesaid countries or regions. This brochure is only delivered to specific person for his or her own reference, is not to be distributed widely as an advertisement.